



## Step-by-Step Restorative Protocol

### ■ SECOND APPOINTMENT

Multi-Unit Abutments, Implant Verification Jig, Abutment-Level Impressions, Wax Rim and Bite



- Remove the patient's healing abutments or existing prosthesis.



- Seat the multi-unit abutments. Refer to case box insert for detailed instructions on how to seat multi-unit abutments.

## ■ SECOND APPOINTMENT

Multi-Unit Abutments, Implant Verification Jig, Abutment-Level Impressions, Wax Rim and Bite



- Seat each section of the jig onto the appropriate multi-unit abutment.



- After seating, make sure the individual sections of the jig do not touch each other.



- Lute the sections together with a flowable composite (e.g., Camouflage® NanoHybrid Composite — Glidewell Direct) or another suitable material.



- Allow the material to flow through and completely around the gaps. Ensure the material is completely cured.

## ■ SECOND APPOINTMENT

Multi-Unit Abutments, Implant Verification Jig,  
Abutment-Level Impressions, Wax Rim and Bite



- Inject VPS impression material (e.g., Capture® Impression Material – Glidewell Direct) under and around the jig to capture the ridge and all anatomical landmarks as for a full denture including full vestibular extensions.



- Using VPS material and the custom tray, take the final impression, ensuring that the guide pins emerge through the appropriate holes in the custom tray.



- Attach the wax rim directly to the multi-unit abutments with the enclosed prosthetic screws. Confirm the wax rim seats passively.

# ■ SECOND APPOINTMENT

Multi-Unit Abutments, Implant Verification Jig,  
Abutment-Level Impressions, Wax Rim and Bite

- Review checklist, making sure to complete each item. Make adjustments to the wax rim as necessary.

<input type="checkbox"/>	<b>Confirm Tooth Shade</b>
<input type="checkbox"/>	<b>Midline</b>
<input type="checkbox"/>	<b>Incisal Edge Position</b>
<input type="checkbox"/>	<b>Lip Support</b>
<input type="checkbox"/>	<b>Smile Line/Occlusal Plane</b>
<input type="checkbox"/>	<b>Vertical Dimension of Occlusion</b>
<input type="checkbox"/>	<b>Phonetics</b>
<input type="checkbox"/>	<b>Bite Registration</b>
<input type="checkbox"/>	<b>Clinical Smile Photo</b>



- Evaluate midline.
- 



- Evaluate incisal edge position and lip support.

## ■ SECOND APPOINTMENT

Multi-Unit Abutments, Implant Verification Jig,  
Abutment-Level Impressions, Wax Rim and Bite



- Establish the occlusal plane.



- Establish vertical dimension of occlusion (VDO).



- Take a bite registration in centric relation.



- Take clinical photos of the smile.

## ■ SECOND APPOINTMENT

Multi-Unit Abutments, Implant Verification Jig, Abutment-Level Impressions, Wax Rim and Bite



- Remove the wax rim and bite registration. **If the patient is wearing a denture, leave the multi-unit abutments in place, cover them with the enclosed multi-unit abutment temporary healing caps and relieve the intaglio surface of the denture to seat over the multi-unit abutments.** *If the patient is wearing a fixed provisional, remove the multi-unit abutments and return them to their original positions on the master cast. Then, reinsert the prosthesis.*

## ■ SECOND APPOINTMENT

Multi-Unit Abutments, Implant Verification Jig, Abutment-Level Impressions, Wax Rim and Bite



Return the case to Glidewell.

*NOTE: Unless the patient is wearing a fixed provisional, once the multi-unit abutments are seated, do not remove them from the patient's mouth.*

**Have questions? Call 866-861-7955 or check out our how-to videos below.**



**Implant Verification Jig**

[bit.ly/UtilizeImplantJig](https://bit.ly/UtilizeImplantJig)



**Adjust Wax Rim**

[bit.ly/AdjustWaxRim](https://bit.ly/AdjustWaxRim)

For more information

[glidewelldental.com/bip](http://glidewelldental.com/bip) ■ 866-861-7955

